

JUNIOR APHTE GEL

JUNIOR APHTE GEL

MEDICAL DEVICE

Tube of 15 ml

JUNIOR APHTE GEL:

- It relieves discomfort and pain quickly.
- It facilitates the healing of canker sores, reducing their duration.
- It contains no alcohol (it does not burn)
- It contains no Parabens
- Without sugar

Thanks to its special composition, **JUNIOR APHTE GEL** acts with an effective barrier effect, enabling a rapid reduction of the local pain and promoting healing of canker sores. The flavor of red fruits makes it pleasant.

INDICATIONS:

Canker sores

Aphthous stomatitis.

COMPOSITION:

Aqua, Xylitol, Glycerin, Rosa damascene petals extract, Xanthan gum, Polycarbophil (1,3%), Sodium hydroxide, Hyaluronic acid sodium salt (0,24%), Pectin, Potassium sorbate (0,2%), Sodium benzoate (0,2%), Stevia, Panthenol, Natural Red fruit Flavors, Aloe barbadensis leaf extract (0,05%).

USE AND DOSAGE:

Apply a small amount of gel on the interested area, after meals, with clean fingers or with a cotton swab. Use two or three times per day (or more, if needed) for a week or until the symptoms have disappeared. Recap after each application. It is recommended not eating and drinking for at least 30 minutes after application, to facilitate the adhesion of the product.

Is not recommended the use of the product for children under 30 months.

STORAGE AND CONSERVATION:

Store below 25 °C. The expiry date refers to the product intact and properly stored. Do not use beyond the expiration date on the package. Use within 30 days after the first opening. The residue or expired product can be disposed as normal household waste.

WARNINGS:

- The product must be used in the oral cavity.
- Do not use in case of individual hypersensitivity to one or more of the components.
- It is recommended to inform your doctor or pharmacist if you are using, have recently used or might use any other medication.
- It is recommended to use at least 2 hours away from medicines and other supplements.
- Do not use during pregnancy and lactation, without consulting your Medical Doctor.
- If symptoms do not improve, seek medical advice.
- Recap after each application.
- Expiry date is for intact product, correctly stored.
- Do not exceed the recommended daily dose.
- Do not use the product if the container is damaged or not intact.
- Do not use beyond the expiry date printed on the package.
- Use the product according to instructions.
- Dispose of properly.
- Keep the product out of reach of children.
- Any serious incidents should be reported to the manufacturer and the relevant local authority, which will provide further advice.

SIDE EFFECTS:

Based on scientific literature, there are no known or predictable adverse effects related to use of the product. In case of undesired effects during use, seek medical advice.

Tube of 15 ml

Distributors










Farmaceutici Procemsa S.p.A.SB
Via Mentana 10 – 10042
Nichelino (TO)



REF 1130902

Symbols description

	Batch code
	Expiry date/ Use-by date
	Manufacturer
	Consult the instructions for use for important cautionary information such as warnings and precautions.
	Consult instructions for use
	Do not disperse after use
	Store below 25°C
REF	Catalogue number
MD	Medical Device
UDI	UDI Symbol

Last version: October 2022

1435330

ADULT APTHE GEL

ADULT APHTE GEL

MEDICAL DEVICE

Tube of 15 ml

ADULT APHTE GEL:

- It relieves discomfort and pain quickly.
- It facilitates the healing of canker sores, reducing their duration.
- It contains no alcohol (it does not burn)
- It contains no Parabens
- Without sugar

Thanks to its special composition, **ADULT APHTE GEL** acts with an effective barrier effect, enabling a rapid reduction of the local pain and promoting healing of canker sores.

The flavor of mint makes it pleasant.

INDICATIONS:

Canker sores

Aphthous stomatitis.

COMPOSITION:

Aqua, Xylitol, Glycerin, Rosa damascene petals extract, Xanthan gum, Polycarbophil (1,3%), Sodium hydroxide, Hyaluronic acid sodium salt (0,24%), Pectin, Potassium sorbate (0,2%), Sodium benzoate (0,2%), Stevia, Panthenol, Aloe barbadensis leaf extract (0,05%), Polysorbate 20, Minth Flavor.

USE AND DOSAGE:

Apply a small amount of gel on the interested area, after meals, with clean fingers or with a cotton swab. Use two or three times per day (or more, if needed) for a week or until the symptoms have disappeared. Recap after each application. It is recommended not eating and drinking for at least 30 minutes after application, to facilitate the adhesion of the product. Is not recommended the use of the product for children under 30 months.

STORAGE AND CONSERVATION:

Store below 25 °C. The expiry date refers to the product intact and properly stored. Do not use beyond the expiration date on the package. Use within 30 days after the first opening. The residue or expired product can be disposed as normal household waste.

WARNINGS:

- The product must be used in the oral cavity.
- Do not use in case of individual hypersensitivity to one or more of the components.
- It is recommended to inform your doctor or pharmacist if you are using, have recently used or might use any other medication.
- It is recommended to use at least 2 hours away from medicines and other supplements.
- Do not use during pregnancy and lactation, without consulting your Medical Doctor.
- If symptoms do not improve, seek medical advice
- Recap after each application.
- Expiry date is for intact product, correctly stored.
- Do not exceed the recommended daily dose.
- Do not use the product if the container is damaged or not intact.
- Do not use beyond the expiry date printed on the package.
- Use the product according to instructions.
- Dispose of properly.
- Keep the product out of reach of children.
- Any serious incidents should be reported to the manufacturer and the relevant local authority, which will provide further advice.

SIDE EFFECTS:

Based on scientific literature, there are no known or predictable adverse effects related to use of the product. In case of undesired effects during use, seek medical advice.

Tube of 15 ml

Distributors










Farmaceutici Procemsa S.p.A.SB
Via Mentana 10 – 10042
Nichelino (TO)



REF 1130896

Symbols description

	Batch code
	Expiry date/ Use-by date
	Manufacturer
	Consult the instructions for use for important cautionary information such as warnings and precautions.
	Consult instructions for use
	Do not disperse after use
	Store below 25°C
REF	Catalogue number
MD	Medical Device
UDI	UDI Symbol

BABY APHTE GEL

BABY APHTE GEL

MEDICAL DEVICE

Tube of 15 ml

BABY APHTE GEL:

- It relieves discomfort and pain quickly.
- It facilitates the healing of canker sores, reducing their duration.
- It contains no alcohol (it does not burn)
- It contains no Parabens
- Without sugar

Thanks to its special composition, **BABY APHTE GEL** acts with an effective barrier effect, enabling a rapid reduction of the local pain and promoting healing of canker sores. The flavor of red fruits makes it pleasant for babies.

INDICATIONS:

Canker sores
Aphthous stomatitis.

COMPOSITION:

Aqua, Xylitol, Glycerin, Rosa damascene petals extract, Polycarbophil (2%), Xanthan gum, Sodium hydroxide, Pectin, Hyaluronic acid sodium salt (0,24%), Potassium sorbate (0,1%), Sodium benzoate (0,1%), Stevia, Panthenol, Aloe barbadensis leaf extract (0,05%), Natural Red fruit Flavors.

USE AND DOSAGE:

Apply a small amount of gel on the interested area, after meals, with clean fingers or with a cotton swab. Use two or three times per day (or more, if needed) for a week or until the symptoms have disappeared. Recap after each application. It is recommended not eating and drinking for at least 30 minutes after application, to facilitate the adhesion of the product.

Is not recommended the use of the product for children under 6 months.

STORAGE AND CONSERVATION:

Store below 25 °C. The expiry date refers to the product intact and properly stored. Do not use beyond the expiration date on the package. Use within 30 days after the first opening. The residue or expired product can be disposed as normal household waste.

WARNINGS:

- The product must be used in the oral cavity.
- Do not use in case of individual hypersensitivity to one or more of the components.
- It is recommended to inform your doctor or pharmacist if you are using, have recently used or might use any other medication.
- It is recommended to use at least 2 hours away from medicines and other supplements.
- Do not use during pregnancy and lactation, without consulting your Medical Doctor.
- If symptoms do not improve, seek medical advice
- Recap after each application.
- Expiry date is for intact product, correctly stored.
- Do not exceed the recommended daily dose.
- Do not use the product if the container is damaged or not intact.
- Do not use beyond the expiry date printed on the package.
- Use the product according to instructions.
- Dispose of properly.
- Keep the product out of reach of children.
- Any serious incidents should be reported to the manufacturer and the relevant local authority, which will provide further advice.

SIDE EFFECTS:

Based on scientific literature, there are no known or predictable adverse effects related to use of the product. In case of undesired effects during use, seek medical advice

Tube of 15 ml

Distributors



Farmaceutici Procemsa S.p.A. SB
Via Mentana 10 – 10042
Nichelino (TO)







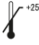





0476



1130888

Symbols description

	Batch code
	Expiry date/ Use-by date
	Manufacturer
	Consult the instructions for use for important cautionary information such as warnings and precautions.
	Consult instructions for use
	Do not disperse after use
	Store below 25°C
	Catalogue number
	Medical Device
	UDI Symbol